BATERIAS DE BOMBAS VOLUNTARIAS/INFUSION

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Batteries Supplied by CSB and Used with Plum Infusion Systems

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ICU Medical identified a potential issue with batteries used in Plum infusion systems. The attached Urgent FSN letter describes the issue and the actions for users to take.

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1. Q What is the issue?

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ICU Medical identified a potential issue with replacement batteries used in Plum Infusion Systems. Due to a manufacturing defect from the battery supplier, affected batteries may experience a loss of capacity earlier in the battery lifecycle than expected and overall battery runtime may decrease earlier in the battery lifecycle than expected.

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If the pump is running on battery power, the user may not have sufficient time to plug the pump into AC power after the Low Battery alarm is activated, which may result in an interruption of therapy. An interruption of therapy may potentially lead to serious patient injury or death, depending on the clinical situation and the type of medication being administered.

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3. Q What products are affected by this notice?

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4. Q Has ICU Medical corrected this issue?

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Yes. ICU Medical worked with CSB to identify and correct the manufacturing defect that led to this issue.

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5. Q Are all Plum batteries affected by this field notice?

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6. Q How do I know if I have affected batteries?

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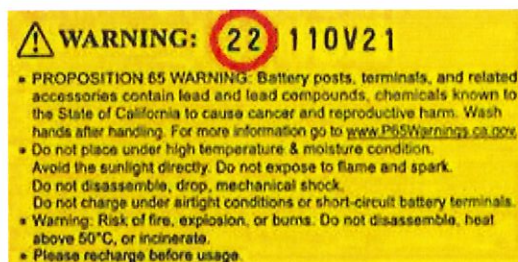
All affected batteries have the CSB logo.

Also the first two characters on the following label indicate the year the battery was manufactured.

If the first two characters are 22 or lower, this issue potentially affects the battery.

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ICU Medical will replace all replacement batteries you have in your possession free of charge. ICU Medical is working to secure as many batteries as possible from our manufacturer as quickly as possible to replace all batteries with all customers.

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Your Plum A+ and Plum 360 pumps are safe to use. Please identify all affected batteries in your possession and ensure all users or potential users of these pumps are immediately made aware of this notification and proposed mitigations.

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No. ICU Medical cannot validate the quality of any alternative battery and will not reimburse any purchases for non-ICU Medical batteries.

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No. Whenever possible, keep the pump plugged into AC power. Before disconnecting the pump from AC power, e.g., to transport a patient, please ensure that the battery is fully charged. Closely monitor the Battery Status Indicator while the pump is disconnected from AC power to help ensure there is sufficient battery capacity to power the pump. Additionally, have a backup pump available when infusing critical medications.

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No. ICU Medical will not come on-site to remove/replace all batteries. ICU Medical will ship replacement batteries to you as quickly as our supply allows.

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12. Q Can I return my pumps to ICU Medical to replace my device batteries?

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No. There is no need to return your Plum A+ or Plum 360 as the pumps are safe to use. ICU Medical will send replacement batteries to you as quickly as our supply allows.

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13. Q Has there been any patient harm due to this issue?

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To date, ICU Medical has received one report of an adverse event potentially related to this issue.

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ICU Medical is issuing this letter to notify you of a potential issue with batteries used in Plum infusion systems. The following information details the issue and the required steps for you to perform.

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Issue:

If a Plum 360 or Plum A+ infusion system is running on battery power, a Low Battery and Depleted Battery alarm should typically activate when thirty minutes and three minutes, respectively, of estimated battery runtime is remaining. In addition, when the pump detects a loss of battery capacity, the pump will display a message to replace the battery. On a Plum 360 pump, the screen will display "Keep Plugged into AC! Service Battery / Replace Pump" and Plum A+ pump will display "Warning: Replace Battery."

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Due to a manufacturing defect from the battery supplier, affected batteries may experience a loss of capacity earlier in the battery lifecycle than expected and overall battery runtime may decrease earlier in the battery lifecycle than expected.

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If an affected battery is not replaced when the pump displays the message to replace the battery, there may be less than thirty minutes of battery runtime after the Low Battery alarm resulting in the Depleted Battery alarm occurring sooner than expected. At this point, the pump stops delivery, alarms Depleted Battery for three minutes, and the pump turns off.

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Potential Risk:

If the pump is running on battery power, the user may not have sufficient time to plug the pump into AC power after the Low Battery alarm is activated, which may result in an interruption of therapy. An interruption of therapy may potentially lead to serious patient injury or death, depending on the clinical situation and the type of medication being administered. **To date, ICU Medical has received one report of an adverse event potentially related to this issue.**

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Affected Product:

Plum A+ and Plum 360 batteries from the supplier, CSB, manufactured before January 1, 2023, are potentially affected by this issue. CSB batteries are identified with the following logo:

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The first two characters on the following label indicate the year the battery was manufactured. If the first two characters are 22 or lower, the battery is potentially affected.

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Required Actions for Users:

There is no need to return or discontinue using your Plum 360 or Plum A+ pumps.

Whenever possible, keep the pump plugged into AC power. Before disconnecting the pump from AC power, e.g., to transport a patient, please ensure that the battery is fully charged. Closely monitor the Battery Status Indicator while the pump is disconnected from AC power to help ensure there is sufficient battery capacity to power the pump. Additionally, have a backup pump available when infusing critical medications.

If a Plum pump displays the Replace Battery alarm mentioned above, continue the infusion with a different pump and remove the pump from clinical use until the battery is replaced. You may replace affected batteries with a new CSB battery until corrected batteries are available. Please do not use a replacement battery with corroded battery terminals.

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1. Identify all affected batteries in your possession and ensure all users or potential users of these pumps are immediately made aware of this notification and proposed mitigations.
2. Complete and return the attached Response Form to EMEA-Quality@icumed.com **within ten days of receipt** to acknowledge your understanding of this notification.
3. **DISTRIBUTORS:** If you have distributed potentially affected products to your customers, please immediately forward this notice to them and ask them to return completed response forms and affected product **to you**. When you have received all completed response forms and affected product from your customers, please complete a SINGLE COMPLETED form with the required details and return to EMEA-Quality@icumed.com

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Follow-up Actions by ICU Medical:

ICU Medical will replace all batteries affected by this issue. We will contact you when replacement batteries are available to schedule the battery replacement.

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For further inquiries, please contact ICU Medical using the information provided below.

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ICU Medical is committed to patient safety and is focused on providing exceptional product reliability and the highest level of customer satisfaction. Thank you for your prompt support on this important matter. We appreciate your cooperation.

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Hofspoor 3
3994 VZ Houten
The Netherlands
www.icumed.com

Sincerely,
Atentamente,

A handwritten signature in blue ink, appearing to read "Corine Broekhuizen", with a long horizontal flourish extending to the right.

Corine Broekhuizen
Director of Quality, ICU Medical BV

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