

**BARRIER®**

BARRIER®  
**Flex**



## **BARRIER®** Laparoscopy set abdo-perineal

- Integrated tape allows adjustment of legging length
- Stretchable adhesive edges to maintain a secure seal when abdomen is inflated during the laparoscopic intervention.
- Integrated tube holders for instrumentation facilitation
- 100% impermeable materials providing optimal patient safety
- Optimal folding for aseptic application, easy for one person to drape

# Draping instruction



1. To start draping, use the adhesive towel as the under buttock drape.



2. Note the direction of the pictogram on the laparoscopy drape. Remove the release paper.



3. Place the drape centered on the patient. Unfold towards yourself and then away from you.



4. See the pictogram of the foot. Take the corner of the legging and unfold over the leg. Do the same for the other leg.



5. Unfold the drape fully towards the head.



6. Drape the perineal cover over the genital area. An option is to use the adhesive tape to attach the end of the drape to the front of the gown of the sterile nurse or surgeon. Ensure that the adhesive film is securely attached to the patient and remove the crepe paper.



7. Draping is then completed.



Scan the QR code for more about Mölnlycke Health Care or to watch a draping instruction video.

Ref. No.	Description	Disp/Trp
694240	4 cellulose towels 18x25 cm 1 OP-tape 9x49 cm 1 tube holder velcro 2.5x30 cm 1 adhesive towel 75x75 cm 1 laparoscopy drape 250/175/270x260 cm, apertures 25x30 cm and 13x24 cm 1 table cover 150x190 cm, 75x190 cm absorbent	5/10
694241	1 mayo stand cover 79x145 cm, 65x85 cm absorbent 4 cellulose towels 18x25 cm 1 OP-tape 9x49 cm 1 tube holder velcro 2.5x30 cm 1 adhesive towel 75x75 cm 1 laparoscopy drape 250/175/270x260 cm, apertures 25x30 cm and 13x24 cm 1 table cover 150x190 cm, 75x190 cm absorbent	5/10
694242	1 mayo stand cover 79x145 cm, 65x85 cm absorbent 4 cellulose towels 18x25 cm 1 OP-tape 9x49 cm 1 tube holder velcro 2.5x30 cm 1 adhesive towel 75x75 cm 1 laparoscopy drape 300/200/270x295 cm, apertures 25x30 cm and 13x24 cm 1 table cover 150x190 cm, 75x190 cm absorbent	5/10

All specified set contents are also available for inclusion in ProcedurePak® trays.


## ProcedurePak components

944200	Laparoscopy drape 250/175/270x260 cm, apertures 25x30 cm and 13x24 cm
944210	Laparoscopy drape 300/200/270x295 cm, apertures 25x30 cm and 13x24 cm

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, Box 13080, SE-402 52 Göteborg, Sweden.  
 T +46 31 722 30 00, F +46 31 722 34 00, [www.molnlycke.com](http://www.molnlycke.com)

The Mölnlycke Health Care, BARRIER™, and ProcedurePak™ names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Copyright 120121

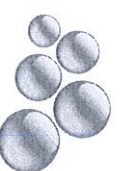

**MÖLNLYCKE**  
HEALTH CARE